

# Thigh/Calf Sleeve

## Instructions For Use

Indication:  
For General  
Support & Compression

Please read these instructions completely and carefully before applying the sleeve. Correct application of the sleeve is important for proper function.

Manufactured by:




Tracy, CA 95304  
Phone: 866.592.0488 • Fax: 209.834.8832

# APPLICATION AND FITTING INSTRUCTIONS

## Thigh Sleeve

Grasp edges of the sleeve at the larger opening. Bring the sleeve onto the leg, pulling it onto the thigh to a comfortable level.

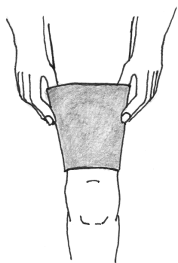

## Calf Sleeve

Bring the sleeve onto the leg so that the calf is centered within the sleeve.



## CARE INSTRUCTIONS

Hand wash in cool water utilizing mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

## WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this brace, consult your healthcare provider immediately.

Neoprene braces and supports should not be worn by individuals with known allergies to neoprene products or susceptibility to dermatitis.

## WARRANTY

Top Shelf guarantees this product to be free of defects in material and construction for a period of six months from date of purchase.