

Universal Shoulder Immobilizer

Instructions for Use

Indication:

Immobilization & support of the shoulder and elbow joints

**Before applying this upper extremity support, please read these instructions completely and carefully.
Correct application is vital to the proper function of this device.**

Manufactured By:


Top Shelf Manufacturing LLC

Toll Free: 1866.592.0488 Phone: 209.834.8185

Fax: 209.834.8832

APPLICATION AND FITTING INSTRUCTIONS

1. Position abdominal belt (A) and secure at front.
2. Secure bicep cuff (B) around arm with attachment straps towards the body.
3. Position shoulder strap (C) opposite affected shoulder, and secure to abdominal belt.
4. Secure wrist/forearm cuff around wrist. Attach bicep cuff to abdominal belt. Adjust as necessary (D).

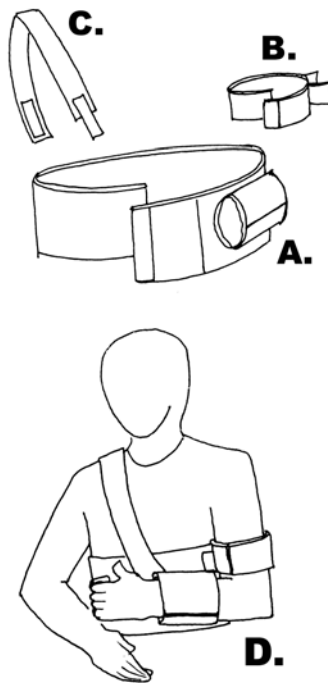

CARE INSTRUCTIONS

Hand wash in cold water using mild soap. Rinse thoroughly and air dry.
If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this support, consult your healthcare provider immediately.

Federal Law (USA) restricts this device to sale by or on the order of a licensed healthcare professional.