

APPLICATION AND FITTING INSTRUCTIONS

- 1.** Place hand in brace with thumb through cutout. The stay (stiff metal insert) should be resting on the palm and must be in-line with the forearm.
- 2.** Wrap and secure closures. For supports with D-Rings, feed straps through D-rings and secure. On mid-strap, pull away from body, wrap
- 3.** around brace and secure.

The brace should be snug and comfortable. Take care not to apply the brace too tightly.

CARE INSTRUCTIONS

Hand wash in cold water using mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this brace, consult your healthcare provider immediately.

Neoprene braces and supports should not be worn by individuals with known allergies to neoprene products or susceptibility to dermatitis.

Federal Law (USA) restricts this device to sale by or on the order of a licensed healthcare professional.

i FIT

Foam Wrist/Forearm

Instructions For Use

Indication:

Wrist Sprains and Strains, Post Surgical Immobilization, Post Fracture Casting

Please read these instructions completely and carefully before applying this brace. Correct application of the brace is important for proper function.

Manufactured By:



TopShelf
Manufacturing, LLC