

APPLICATION AND FITTING INSTRUCTIONS

- 1.** Loosen straps and laces, place hand in brace. The stay (stiff metal insert) must be resting on the palm and be in-line with the forearm.
- 2.** Wrap and secure closures.
- 3.** The brace should be snug and comfortable. Take care not to apply the brace too tightly.

CARE INSTRUCTIONS

Hand wash in cold water using mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this brace, consult your healthcare provider immediately.

Federal Law (USA) restricts this device to sale by or on the order of a licensed healthcare professional.

i FIT

Lace-Up Wrist Support

Instructions For Use

Indication:
Carpal Tunnel Syndrome, wrist
immobilization, painful hand or wrist

Please read these instructions completely and
carefully before applying this brace.
Correct application of the brace is
important for proper function.

Manufactured By:
 **TopShelf**[™]
Manufacturing, LLC