

topshelforthopedics.com  
Phone: 866.592.0488 • Fax: 209.834.8832  
Tracy, CA 95304  
1851 East Paradise Road, Suite A



## Universal Gel Wrist

### Instructions for Use

Please review these instructions completely and carefully before applying this brace.







Correct application of the brace is important for proper function.



# Universal Gel Wrist Instructions For Use

Please review these instructions completely and carefully before applying this brace. Correct application is important for proper function of the brace.

Indications: Carpal Tunnel Syndrome, Wrist Immobilization, Painful Hand & Wrist.

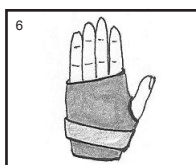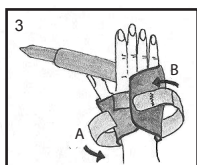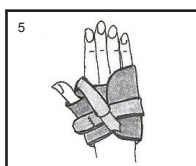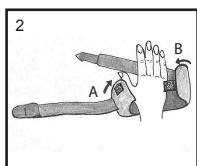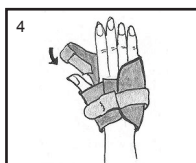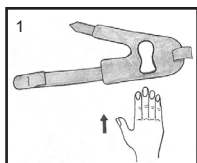

**Use & Care:** Hand wash in cold water using mild soap, rinse thoroughly. Air dry only.

**Note:** If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material.

## **Warning:**

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

## **Warning:**

Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene.

## **Warranty:**

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from the date of purchase.

## **Caution:**

Federal Law (U.S.A.) restricts this device to sale by or on the order of a licensed healthcare professional.